VERMEIREN

Illico

INSTRUCTION MANUAL
MANUEL D'UTILISATION
GEBRUIKSAANWIJZING
GEBRAUCHSANWEISUNG
MANUALE DI ISTRUZIONI
ANUAL DE INSTRUCCIONES
INSTRUKCJA OBSŁUGI
NÁVOD K OBSLUZE

EN

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Manual de instrucciones

Cama de cuidados

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Beds and detachable bed boards/mattress support platforms with powered adjustment – ISO 9999: 18 12 10

This page is intentionally left blank

Contents

Preface	2
1 Your product	3
1.1 Description	3
1.2 Optional parts	4
1.3 Expected lifespan.....	4
1.4 Declaration of conformity	5
2 Before use	5
2.1 Intended use	5
2.2 General safety instructions	5
2.3 Symbols on the nursing bed	6
3 Using your nursing bed	7
3.1 Assembling/disassembling, transport and storage.....	7
3.2 Installation.....	7
3.3 Using the handset.....	8
3.5 Adjusting leg section manually.....	10
4 Maintenance	11
4.1 Points of maintenance	11
4.2 End of use.....	11
5 Troubleshooting	12
6 Technical specifications	13

Preface

Congratulations! You are now owner of a Vermeiren nursing bed!

This nursing bed is made by qualified and committed personnel. It is designed and produced according to high quality standards guarded by Vermeiren.

Thanks for your trust in the products of Vermeiren. To support you on the use of this bed and its operating options, this manual is offered. Please read it carefully; it will help you to get familiar with the operation, capabilities and limitations of your bed.

If you still have questions after reading this manual, do not hesitate to contact your specialist dealer. He/she will be glad to help you on this subject.

Important note

To ensure your safety and to prolong the lifetime of your product, please take good care of it and have it checked and/serviced on regular base.

The warranty of this product is based on normal use and maintenance as described in this manual. Damage to your product caused by improper use or lack of maintenance will cause the warranty to lapse.

This manual reflects the latest product developments. Vermeiren has the right to implement changes to this type of product without being imposed to any obligation to adapt or replace similar products previously delivered.

Pictures are used to clarify the instructions in this manual. Details of the depicted product may deviate from your product.

Information available

On our website <http://www.vermeiren.com/> you will always find the most recent version of following information. Please consult this website regularly for possible updates.

Visually impaired people can download the electronic version of this manual and have it read out by means of a text-to-speech software application.

- This manual
For user and specialist dealer

- EC declaration of conformity
For user and specialist dealer

- Installation manual (Instructions for assembling and disassembling°
For specialist dealer

- Service manual (Beds)
For specialist dealer

1 Your product

1.1 Description

The ILLICO nursing bed is easy to assemble or disassemble which makes it suitable for temporary use.

i Before using your nursing bed, check the technical details and limits of intended use, see chapter 6.

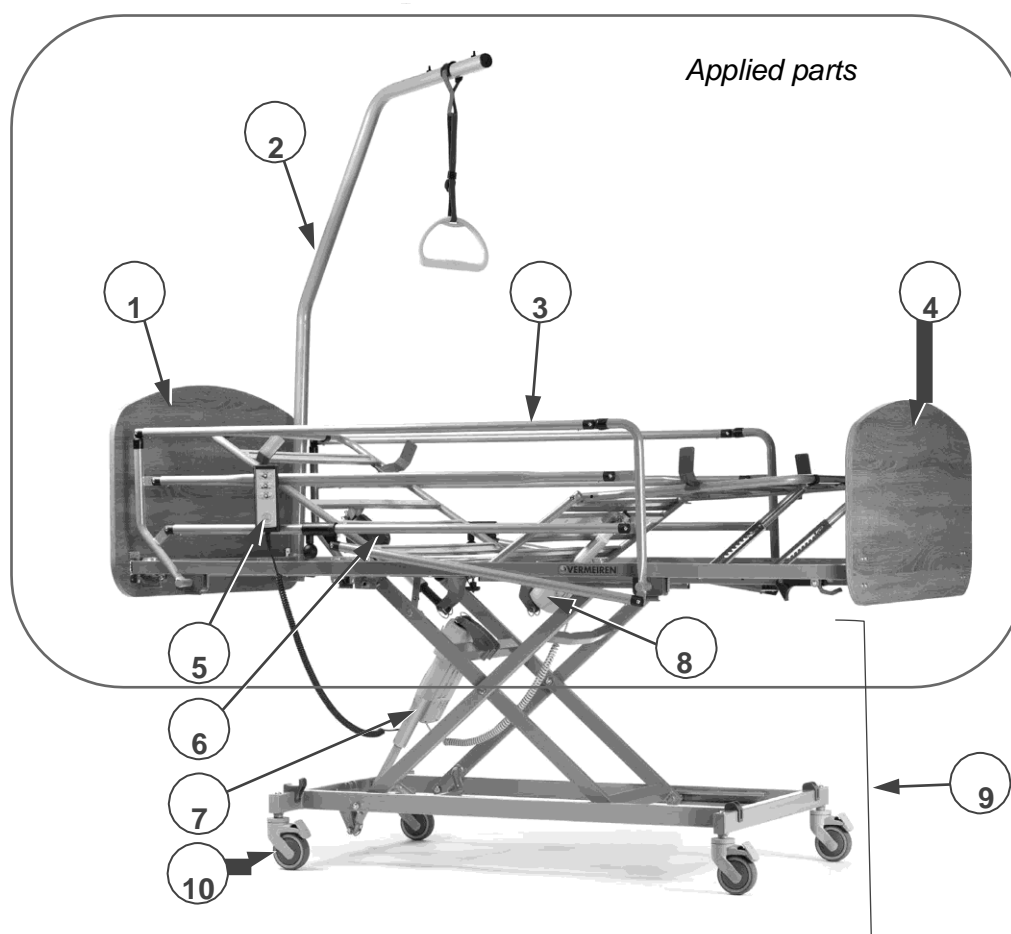
Figure 1 Important parts; bed in highest position

- | | |
|----------------------------------|---|
| 1. Headboard | 7. Motor of mattress support |
| 2. Lifting pole / Trapeze handle | 8. Motor of knee-bend system (optional) |
| 3. Side rail, foldable | 9. Undercarriage |
| 4. Footboard | 10. Castor wheel with brake |
| 5. Handset | |
| 6. Motor of head section | |

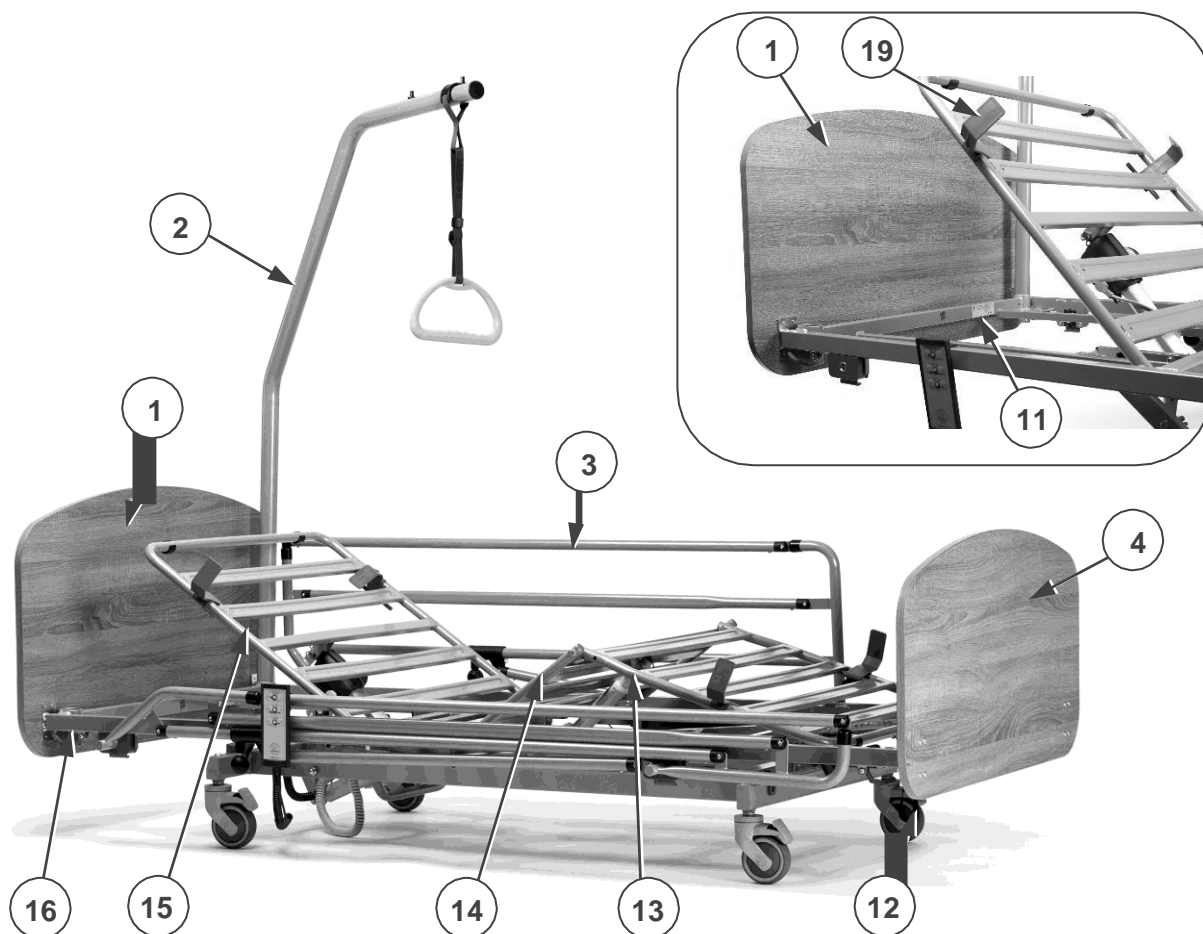
Figure 2 Important parts; bed in lowest position

- | | |
|--|---|
| 1. Headboard | 13. Lower-leg section |
| 2. Lifting pole / Trapeze handle | 14. Upper-leg section |
| 3. Side rail, foldable | 15. Head section |
| 4. Footboard | 16. Mattress support, comprising items (13+14+15) |
| 11. Location of identification plate (on bed frame, at head end) | 19. Mattress stopper |
| 12. Strap | |

1.2 Optional parts

Your nursing bed may be equipped with following parts:

- Extra motor (8) for electric operation of the knee bend mechanism;
- Headboard, footboard, side rails are exchangeable;
- Mattress frame with supports in plastic, instead of metal.

For more information, contact your specialist dealer.

1.3 Expected lifespan

The expected lifespan of this product is reached when used according to the instructions in this manual and the service manual. The effective lifespan depends on use, care and maintenance.

1.4 Declaration of conformity

This product is certified for CE and is classified as a Medical device, Class II. The CE mark indicates the conformity of the product with standards for protection of health, safety and environment applied on products traded within the European economic area.

A copy of the Declaration of conformity is available via our website:

<http://www.vermeiren.com/>.

2 Before use

2.1 Intended use

In this paragraph a brief description of the intended use of your nursing bed is given. Additionally, relevant warnings are added to the instructions in the other paragraphs. In this way we would like to make you aware of the possible misuse that may appear.

- This nursing bed is classified as a Class II electrical product and is suitable for indoor use only.
- This nursing bed is designed and produced solely as resting place, to support one (1) person with a maximum weight of 140kg.
- The nursing bed is designed for home-care use according to IEC standard 60601-2-52, application environment 4.
- Only use a suitable mattress on this bed, see chapter 6 for the details.
- Only use accessories and spare parts approved by Vermeiren.

2.2 General safety instructions

 **CAUTION** Risk of injuries and damage

Follow the instructions in this manual and observe the safety directives. If not, you or bystanders may get injured, or your nursing bed may get damaged.

- Your bed has been tested on electromagnetic compability and complies to the standard. Despite this compability is it possible that the operation performance is influenced by electromagnetic fields like those of cell phones, power generators of high power energy sources.

 **CAUTION** Risk of damage

- It is possible that the electronics of the bed influences other electronic devices. For this reason, it is advised to check your bed on damage or wear, see chapter 4
- Do not add any mechanical or electrical devices to the bed.
- Use the bed only on flat, level, solid floors with all four wheels touching the ground.
- Make sure that during the operation of the bed, no objects get entrapped between the moving parts of the bed.

2.3 Symbols on the nursing bed

The symbols in following list are applicable for your nursing bed. Missing symbols can be found in the relevant ISO standard (ISO 7000, ISO 7001 and IEC 417).


Maximum weight of the user (in kg)

Maximum safe working load (in kg)

Maximum mattress thickness (in mm)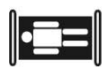

Mattress dimensions (in cm)

Type designation

Risk of entrapment

3 Using your nursing bed

3.1 Assembling/disassembling, transport and storage

The bed will be assembled by your specialist dealer according to the instructions on our website.

For transport, all parts are collected on a transport trolley. Attach the transport trolley and bed parts firmly to the vehicle, so they won't move during transport.

If not in use, store the bed at a dry location, see chapter 6 for the details.

3.2 Installation

 CAUTION Risk of injuries and damage

Make sure that the bed is moved without any person in it.

- If not yet assembled, assemble the bed according to the instructions on the website.
- If the bed is in use, make sure that all covers and sheets are within the side rails.
- Unplug the power cable from the socket and place the cable end on the bed.
- Release all brakes of the castor wheels, see [A] in next figure.
- Roll the empty bed in place.
- When in position, apply the four brakes, see [B] in next figure.
- Connect the power cable of the bed to the 220/230V AC power socket

Releasing brakes [A]:
Use your foot to press the top of the brake lever backwards.

Applying brakes [B]:
Use your foot to press the brake lever down towards the castor wheel.

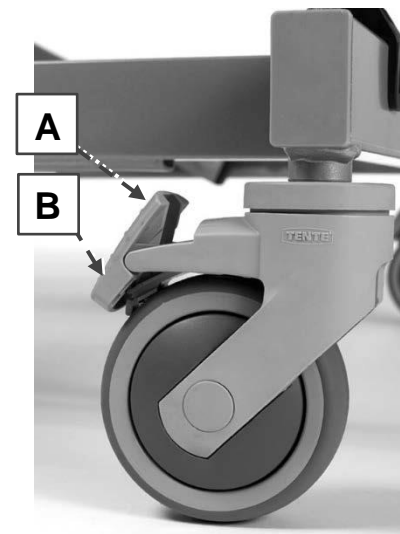
Figure 3 Castor wheel with brake

3.3 Using the handset

CAUTION

Risk of damage

Make sure that no objects get entrapped between the moving parts.

i

- The user does not need any skills, training or knowledge to operate the bed safely.
- The bed parts move slow to prevent any entrapment of limbs.
- Maybe some of next buttons do not respond when you press it. This is since they are blocked for use, by nurse or caregiver according to your medical condition.

3.3.1 Use by patient

The nursing bed is delivered with a handset to operate the movable parts.

- To operate the bed, press and hold the arrow (UP / DOWN) of one of buttons (a – c).
- To stop the movement, release the button.

Operating items:

- (a) Head-section angle (UP / DOWN)
- (b) Mattress support (UP / DOWN)
- (c) Knee bend (UP / DOWN) (optional)
- (d) Button lock (rotatable switch)
- (e) Lock key for button lock (d)

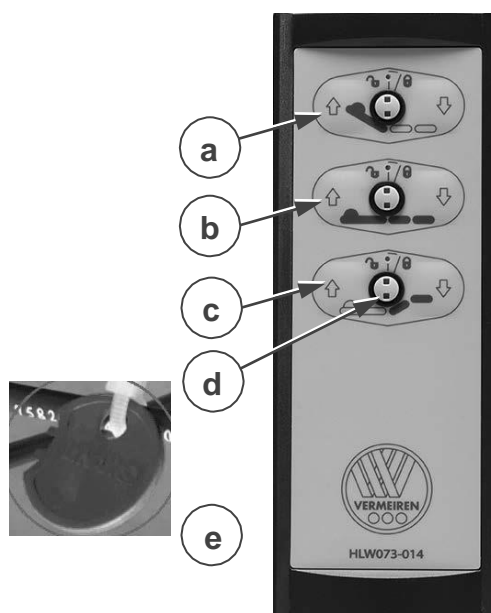
Figure 4 Handset and key to lock

3.3.2 Locking function

CAUTION

Risk of injuries

The lock key may ONLY be used by authorised medical personnel.


To prevent that the patient undergoes unintentional movements, the buttons of the handset can be locked by button lock (d), which is a rotatable switch.

To lock/unlock:

- Use the special key delivered with the bed and turn button lock (d)
 - clockwise to lock or
 - counter clockwise to unlock.

Indicator (f) in the middle of the key turns

- green when the button is unlocked;
- yellow when the button is locked.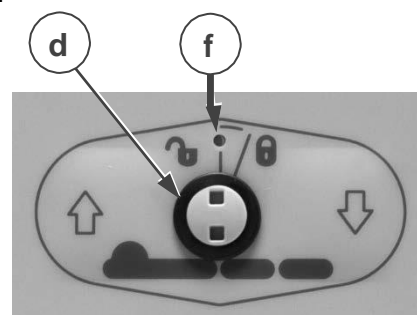


3.4 Getting up / going to bed

 CAUTION Risk of falling

In case you cannot perform the transfer in a safe manner, ask someone to assist you.

Going to bed

1. Check whether the brakes of the castor wheels (10) are applied, see Figure 3.
2. Adjust the height of the bed with the handset; with button (b), see Figure 4.
3. Fold side rail (3): pull the black, spherical knob (17) to unlock the side rail and fold the rail down.
4. Position you as close as possible to the bed.
5. Transfer to the bed. Use the lifting pole with trapeze handle (2) to help move yourself on the bed.
6. Unfold the side rail (3): pull the rail up till its locked.

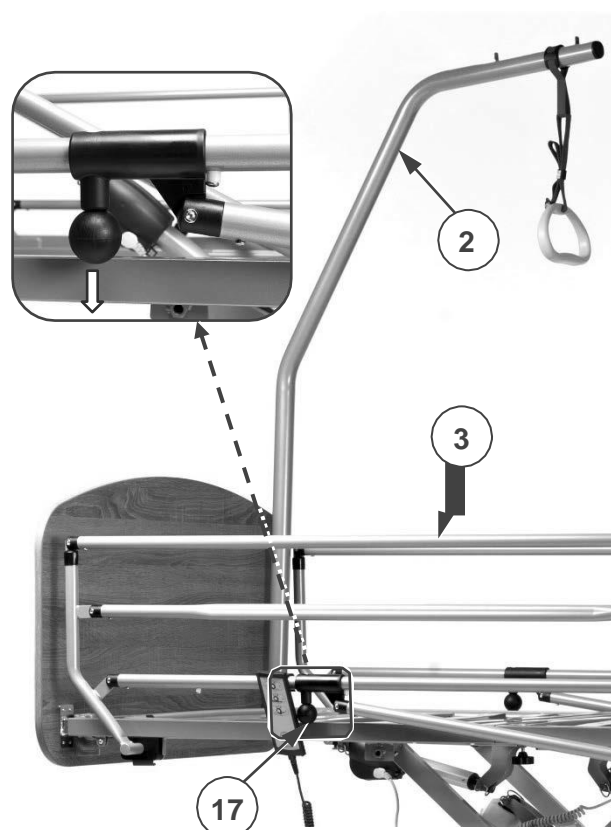Getting up from the bed

1. Use button (a) of the handset to lower the head section till it is completely down, see Figure 4.
2. Use button (c) to lower the knee-bend system completely (if electric operated).
3. Use strap (12) to lower the leg section, see §3.5.
4. Fold the side rail (3): pull the black, spherical knob (17) to unlock the side rail and fold the rail down.
5. Adjust the height of the bed with button (b) on the handset; see Figure 4.
6. Turn yourself to sitting positions with your legs outside the bed. To move yourself on the bed, use the lifting pole with trapeze handle (2).
7. Transfer out of the bed.

Figure 5 Folding/unfolding side rail

3.5 Adjusting leg section manually

**CAUTION**

Risk of injury by entrapment or damage to the bed

- Make sure that no bystanders or objects get trapped between the moving parts.
- Only use strap (12) at the foot end to adjust the height of the leg section.

Raising legs/feet

- To adjust the height of feet and legs, grip the strap at the foot end of the bed.
- Pull the strap up to raise the leg section of the mattress support to the desired height. When you stop pulling, the system stays locked at a notch of rastomat (18).

Lowering legs/feet

- To lower, pull the strap to the highest position to unlock rastomat (18).
- Hold the strap and move the leg section slowly down.

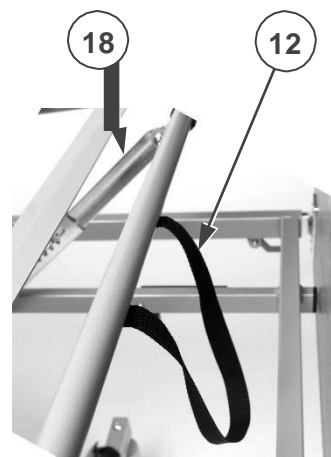
Figure 6 Adjusting knee height

4 Maintenance

Your nursing bed requires minimum maintenance, but it is advisable to inspect the nursing bed regularly in order to keep it trouble-free for years.

4.1 Points of maintenance

CAUTION Risk of damage

- Servicing, repairs and disinfection should only be undertaken by authorized and trained persons.
- Keep handset, motors and connection points clean and protect them from moisture.

i

- The last page of this manual contains a registration form for the specialist dealer to record each service.
- Contact your specialist dealer to agree to a common timetable for inspection/maintenance/repair.
- To find a service facility or specialist dealer near you, contact the nearest Vermeiren company division. A list of Vermeiren company divisions can be found on the cover page.

Once a year / for every new user

This nursing bed needs to be serviced and disinfected before passing on to a new user, but at least once a year. Contact your specialist dealer for more information.

Once a month

Visually inspect next points regularly:

- Keep your castor wheels free of wires, hair, sand and carpet fibres.
- Check the working of the brakes. If the bed can be moved while the brakes are applied, you should check wheels and brakes for wear, damage of contamination like oil, water...
- Handset, motors, connection points and cables: check for damage like electrical wires that are frayed, broken or that become exposed.
- Check if all parts are well secured.
- Check all parts for damage or wear. Contact your specialist dealer for possible repairs or part replacements.

Once a week

Cleaning:

- Use a soft cloth or vacuum cleaner to remove dust.
- In case of dirt, empty the nursing bed and remove the mattress. Wipe all parts with a damp cloth (not drenched). If necessary, use a mild soap, suitable for varnishes and synthetics.

If stored

Make sure that the bed is stored dry to prevent mould or rust, see details in chapter 6.


4.2 End of use

At end of life, you need to dispose your nursing bed according to the local environmental legislation. The best way to do so, is to disassemble the nursing bed to facilitate the transport of recyclable parts.

5 Troubleshooting

Even if you use your nursing bed properly a technical problem may occur despite this. In this case, contact your local specialist dealer.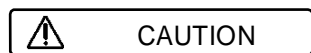

Risk of injuries and damage to the nursing bed

- NEVER repair your nursing bed yourself attempting to solve the problem.

Next symptoms may indicate a problem. Therefore, contact your specialist dealer always if you spot any of following deviations:

- Strang sound;
- Frayed/damaged power cables;
- Cracked or broken connectors;
- Jerky movements of frame sections;
- Damaged or broken castor wheels;
- Nursing bed is powered, but the handset does not work.

6 Technical specifications

The technical details below are only valid for this nursing bed, at standard settings and optimal ambient conditions. Take these details into account during use.

The values are no longer applicable if your nursing bed has been modified, damaged or suffers from severe wear.

Table 1: Technical specifications

Brand	Vermeiren
Address	Vermeirenplein 1/15, B-2920 Kalmthout, Belgium
Type	Nursing bed
Model	Illico

Description	Dimensions
Max. weight of user	140 kg
Length	2080 mm
Height	783 mm - 1313 mm
Width	1024 mm
Height of mattress support, adjustable	Continuously from 298 to 828 mm (without mattress)
Total mass	85,3 kg (excl. side rails, lifting pole and triangle)
Total mass of heaviest part	31 kg
Max. angle head section, adjustable	70°
Max. angle for upper leg	36,8°
Max. angle for lower leg	-17,2°
Max. safe working load	170 kg
Nominal load of lifting pole	75 kg
Mattress specifications	Min. RG 35, thickness 120 mm, 2000 x 900 mm, max. 20 kg
Clearance below bed for patient hoist	150 mm
Castor wheels (4 pieces)	Ø 100 mm, all wheels with brake
Motor of undercarriage	Linak LA40, IPX6
Motor of head section	Linak LA27, IPX6
Motor of knee-bend system (optional)	Linak LA27, IPX6
Handset	Linak HL7X045-01, IPX4
Transformer	Linak CA40, IPX6
Nominal voltage, transformer	U _{in} : 100 - 240 V AC, 50/60 Hz I _{in} : max. 2.5 A
Period switched on	10%, Maximum 2 minutes then 18 minutes for non-use
Noise level	~ 48 dB(A)
Class / Kind of protection	Protection class II, application part Type B / IPX4

Description	Dimensions
Temperature to use	+5 °C to +40 °C
Storing temperature	-10°C to +50°C
Humidity to store / use	20% / 90%
We reserve the right to introduce technical changes. Measurement tolerance ± 15 mm / 1,5 kg / 1,5°.	

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Service registration form

This product (name):

was inspected (I), serviced (S), repaired (R) or disinfected (D):

By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:
By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:	By (stamp): Kind of work: I / S / R / D Date:

Belgium

Vermeiren N.V.

Vermeirenplein 1 / 15

B-2920 Kalmthout

phone: +32(0)3 620 20 20

fax: +32(0)3 666 48 94

website: www.vermeiren.be

e-mail: info@vermeiren.be

France

Vermeiren France S.A.

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59133 Phalempin

phone: +33(0)3 28 55 07 98

fax: +33(0)3 20 90 28 89

website: www.vermeiren.fr

e-mail: info@vermeiren.fr

Italy

Vermeiren Italia

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I-20020 Arese MI

phone: +39 02 99 77 07

fax: +39 02 93 58 56 17

website: www.vermeiren.it

e-mail: info@vermeiren.it

Poland

Vermeiren Polska Sp. z o.o.

ul. Łączna 1

PL-55-100 Trzebnica

phone: +48(0)71 387 42 00

fax: +48(0)71 387 05 74

website: www.vermeiren.pl

e-mail: info@vermeiren.pl

Czech Republic

Vermeiren ČR S.R.O.

Nádražní 132

702 00 Ostrava 1

phone: +420 596 133 923

fax: +420 596 121 976

website: www.vermeiren.cz

e-mail: info@vermeiren.cz

Germany

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D-40472 Düsseldorf

phone: +49(0)211 94 27 90

fax: +49(0)211 65 36 00

website: www.vermeiren.de

e-mail: info@vermeiren.de

Austria

Vermeiren Austria GmbH

Schärdinger Strasse 4

A-4061 Pasching

phone: +43(0)7229 64900

fax: +43(0)7229 64900-90

website: www.vermeiren.at

e-mail: info@vermeiren.at

Switzerland

Vermeiren Suisse S.A.

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3645 Gwatt (Thun)

phone: +41(0)33 335 14 75

fax: +41(0)33 335 14 67

website: www.vermeiren.ch

e-mail: info@vermeiren.ch

Spain / Portugal

Vermeiren Iberica, S.L.

Carratera de Cartellà, Km 0,5

Sant Gregori Parc Industrial Edifici

A 17150 Sant Gregori (Girona)

phone: +34 972 42 84 33

fax: +34 972 40 50 54

website: www.vermeiren.es

e-mail: info@vermeiren.es

Manufacturer

Vermeiren GROUP

Vermeirenplein 1 / 15

B-2920 Kalmthout

phone: +32(0)3 620 20 20

fax: +32(0)3 666 48 94

website: www.vermeiren.com

e-mail: info@vermeiren.com

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All rights reserved, including translation.